TOYOTA

TOYOTA HIACE 3500 CC

AMBULANCE TOYOTA HIACE 2026

PASMED 

TOYOTA

TOYOTA HIACE 3500 CC

AMBULANCE TOYOTA HIACE 2026

PASMED 

TOYOTA

TOYOTA HIACE 3500 CC

AMBULANCE TOYOTA HIACE 2026

PASMED 

TOYOTA HIACE 3500 CC

AMBULANCE TOYOTA HIACE 2026

TOYOTA HIACE 3500 CC

AMBULANCE TOYOTA HIACE 2026

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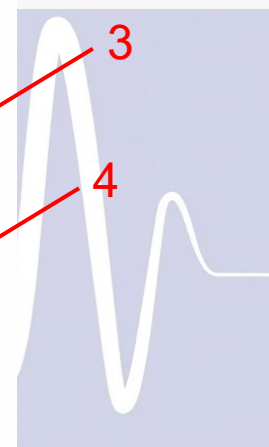
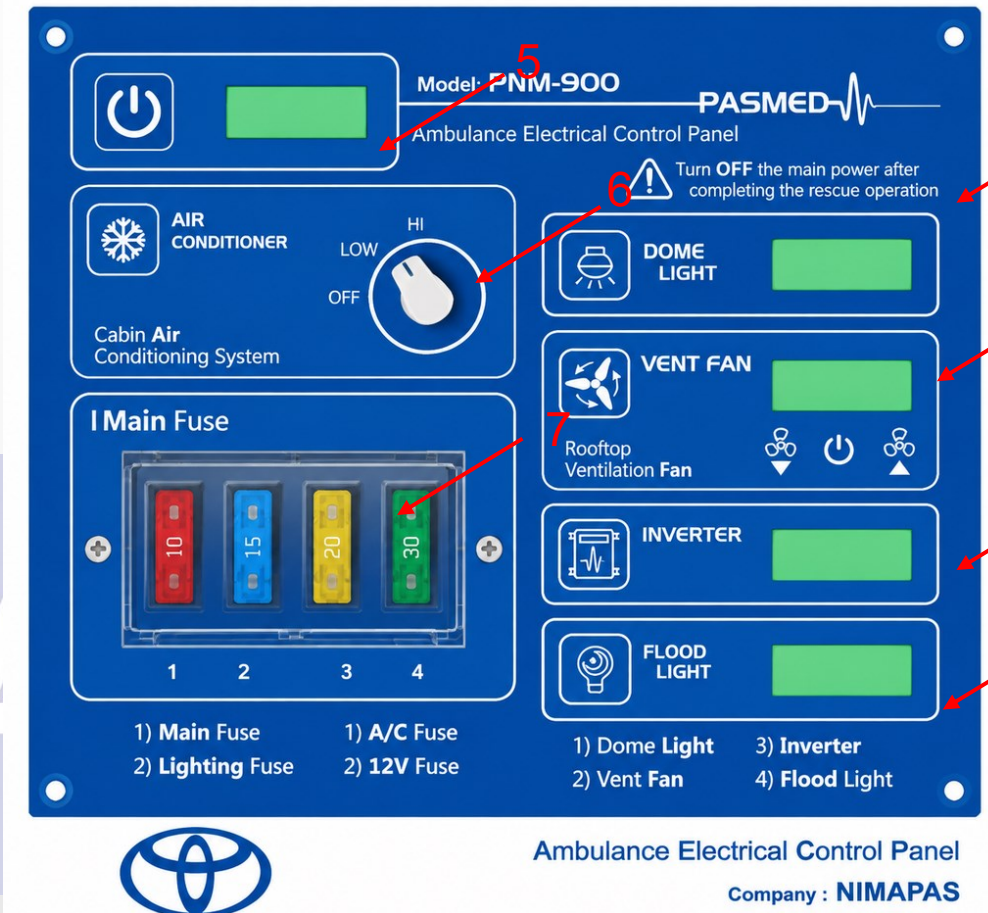
18- Vital signs monitor

19- Syringe pump

20- Location for convertible stretcher (chair-stretcher)

PASMED

Ambulance Control Panel:

- | | |
|--------------------|------------------------------------|
| 1- Ceiling light | 6- Heater and A/C system |
| 2- Ventilation fan | 7- Patient compartment fuse system |
| 3- Inverter | |
| 4- Floodlight | |
| 5- On/Off | |

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Safety and Maintenance Guidelines:

Pre-dispatch Checks:

Daily inspection and full preparation of the ambulance include checking the equipment. This review ensures that everything is in its proper place and readily accessible. To this end, you must check the following items in the ambulance:

Check the following items daily: cooling system, fuel level, engine oil level, transmission fluid level, brake fluid, engine belts, all interior and exterior lights, windshield wipers and fluid, horn, tire pressure (including the spare), siren and roof-mounted warning lights, heating and cooling systems, ventilation system, doors, and all portable and fixed two-way radios. Also, check medical supplies—including oxygen cylinders, resuscitation kits, splints, dressings and bandages, long backboards, other patient immobilization equipment, and emergency childbirth kits—on a daily basis. All battery-operated equipment, such as suction units and defibrillators, must be tested daily. During these checks, verify whether the equipment is functioning correctly, and ensure that supplies are clean, sterile, and adequately stocked.

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Preparation phase:

Stock the ambulance with medical supplies and equipment based on necessity and anticipated needs.

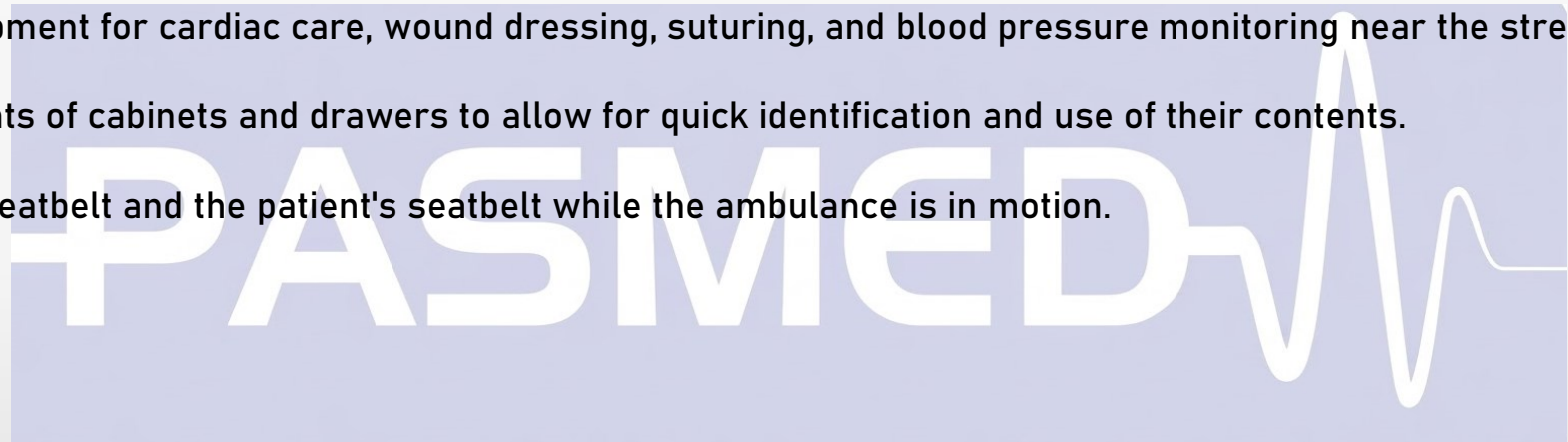
Prioritize items essential for patient care during life-threatening situations, such as equipment for airway management, artificial respiration, and oxygen delivery.

Place these items in the cabinet next to the technician's seat so they are easily accessible.

Position equipment for cardiac care, wound dressing, suturing, and blood pressure monitoring near the stretcher.

Label the fronts of cabinets and drawers to allow for quick identification and use of their contents.

Fasten your seatbelt and the patient's seatbelt while the ambulance is in motion.

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Warranty and After-Sales Service:

Dear Customer: 

The registered trademark belongs exclusively to Nima P.A.S. Company. Products bearing this mark are covered by Nima P.A.S. after-sales services for the duration of the warranty period, calculated from the date of the sales invoice. Thanks to the high quality of the products and services provided, this trademark currently enjoys significant prestige in Iran.

Nima P.A.S. Company is equipped to address all customer-related inquiries in accordance with international standards, specifically the ISO 1002:2004 standard for customer complaint handling systems.

Steps to access Nima P.A.S. Company's after-sales services:

1- Contact Nima P.A.S. Company Customer Service:

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2- Scheduling the Service Appointment:

All requests are reviewed by specialists at the earliest opportunity. The customer will be contacted to schedule an appointment or arrange for the product to be sent to the head office or factory.

3- Actions Taken by the Technical Customer Service Unit:

Experienced specialists in the Technical Customer Service Unit will conduct a thorough inspection and troubleshooting of the product. They will promptly resolve the issue reported by the customer as well as any other potential defects. Subsequently, the product will undergo performance evaluations and specialized testing by the company's Quality Control Unit to ensure it is returned to the customer with its original quality standards.

4- Product Return:

The Customer Affairs Department will make the necessary arrangements to return the product to the customer as soon as possible.

5- Quality Assessment of Services Provided:

Your satisfaction with the quality and speed of the services provided by the After-Sales Service Department is a priority for Nima P.A.S. Company.

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Business Hours:

Dear Customers,

Nima P.A.S. Company is available to address all your inquiries from Saturday to Thursday, between 8:30 AM and 5:00 PM.

We welcome your comments, suggestions, and feedback.

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Sincerely, Nima P.A.S. Company (Trading, Equipping, and Custom Outfitting)

PASMED